Manufacturer Information on Reprocessing of Medical Devices according to
DIN EN ISO 17664:2004

Manufacturer: HELMUT ZEPF Medizintechnik GmbH
Obere Hauptstrasse 16-22
78606 Seitingen-Oberflacht, Germany
Tel.: +49 (0) 7464 / 98 88 -0
Fax: +49 (0) 7464 / 98 88 -88
E-Mail: info@zepf-dental.com

Valid as of: 05/2011 (Rev: 2)

Product: HF Electrodes

REF 452 403 – 452 416

Warning notices: no special notices

Limitation of reprocessing: min. 75 reprocessings

Place of Handling: Remove surface contamination with a tissue. A surface cleaning and rinsing should be carried out immediately after every single use. Otherwise, particles or dried secretion could adhere to the surface and make a later cleaning and sterilization more difficult or even impossible.

Storage and Transport: Touch electrodes at their insulated shafts only, do not touch the wires. Exception: Electrodes with adjustable wire.

Preparations for Cleaning: If an instrument can be disassembled, it should be cleaned in disassembled state. (Note: Keep all parts together and protected from being misplaced, do not mix with other or similar devices of other manufactures.)

Cleaning: Only use cleansers according to the instructions of the respective manufacturer. Do not use acid cleansers. Ideal are agents with neutral pH value. Enzymatic cleansers should be used for the removal of protein rests. Don't use any abrasive cleaners. Only use soft brushes. Rinse and brush all tubes. Rinse off cleanser thoroughly with distilled water after cleaning. Dry electrodes immediately. Do not exceed 93° C for mechanical cleaning.

Disinfection: Use disinfecting solution suitable for synthetics/plastic according to the instructions of the respective manufacturer. Remove disinfecting solution thoroughly under running water. Dry electrodes immediately. Do not exceed 93° C for mechanical disinfection.

Maintenance: No special requirements.

Packaging: Standardized packing material for sterilisation can be used (recommendation).

Sterilization: Vapor sterilization at 134° C for 20 minutes

Control/Functional Check: Sight check on damages, wear, deformation.

Storage: Electrodes should be stored in a clean, dry place, either single-packed or in a box with single compartments. In case you store the electrodes in drawers, protect the tips with cloth, gauze or tubes.

The a.m. instructions have been validated as SUITABLE for preparation of a medical device and its re-use by the medical device manufacturer. It is the reprocessor's responsibility that the actual reprocessing achieves the required results with the equipment and materials applied and personnel involved in the reprocessing site. Normally, validation and routine control are necessary. Furthermore, the reprocessor should evaluate any deviation from the provided instructions regarding efficiency and possible adverse consequences.

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GB

Electrodes for High Frequency Radio Frequency Surgical Units

Important information – Please read prior to use!

Attention!

Charge Maximale: 230V / 50Hz / 80VA, 50 Watt – 1 kOhm.

Handling: All electrodes should be handled with utmost care during transport, cleaning, maintenance, sterilization and storage. This is especially important for fine wire and loop electrodes. Do not expose electrodes to acid or aggressive cleansers as this may cause corrosion and functional impairment. Improper handling and care as well as misuse of the electrodes may result in premature signs of wear. Persons using these electrodes should be knowledgeable in regard to high frequency/radio frequency devices and their electrodes.

The electrode wires are made of fine, surgical tungsten that wears and tears with use. In order to prolong the life span of the electrodes, please observe the following recommendations:

1. Touch electrodes at their insulated shafts only, do not touch the wires. Exception: Electrodes with adjustable wire.
2. Bend electrodes at their flexible insulated shafts only.
3. Insert electrode in the handpiece all the way, no metal part of the electrode should be visible.
4. Select a setting that allows for the electrode to slide through the tissue without drag or pull and without spark formation.
5. Should the electrode wire buckle while cutting, the unit setting is not chosen correctly or the wire worn out. Correct the setting or replace the electrode. Exception: The wire of electrodes with adjustable wire can be pulled out and the used part can be cut off.
6. Remove tissue residue that collects on the wire during an operation as follows:
 - 6.1. Leave the electrode in the handpiece.
 - 6.2. Place two gauze pads or swabs on top of each other, fold them in the middle and soak them with water.
 - 6.3. Place electrode between the folded gauze pads/swabs.
 - 6.4. Activate high frequency/radio frequency surgical unit for 1 – 2 seconds while moving the electrode back and forth with low finger pressure. The sizzling sound you hear is caused by evaporating water molecules which clean the electrode. Repeat this procedure if necessary.

Every surgical instrument is to be cleaned thoroughly and disinfected after each usage. Appropriate cleaning, inspection and maintenance will ensure correct function of the instruments. All instruments are to be autoclaved before using them. Correct cleaning and maintenance will extend the lifetime of the instrument.

For best results the cleaning and rinsing must take place immediately after each usage. Failure to prompt cleaning may result in adherent particles or dried secretions which may resist cleaning and complicate future sterilization. Instruments must be completely free of any contaminant.

The vibrations of ultrasound cleaners may loosen the electrode wires. The electrodes must be allowed to cool down to room temperature prior to use.

Prior to first use: After you are familiar with the correct dosage of the device and you are confident about the flawless condition of the electrode, you should check the cut movement before the first usage. Clean and sterilize (up to 134 °C / 300 °F) electrodes prior to their first use.

Examination and functional test: Examine electrodes for damage prior to every use. Electrodes with defective wires, shafts or coatings are not repairable and should be discarded.

Use of electrodes: High frequency / radio frequency surgery can carry a significant risk if it is performed incorrect, improper or careless. Avoid a contact of the electrode with non-insulated instruments. Persons working with high frequency / radio frequency devices should possess the necessary knowledge about such units. Do **not** use high frequency / radio frequency surgical units on patients with pace makers or in the presence of flammable liquids or anaesthetic agents. Observe the recommendations of the device manufacturer.

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HAGER & WERKEN GmbH & Co. KG