# LaserHF® PARO-PDT

---

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---

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## GB Directions for use

---

### Photo-activated disinfection of periodontal pockets and for treatment of peri-implantitis using a viscous solution

Composition of **LaserHF Para-PDT** solution: Medical grade tolonium chloride powder, water, sodium phosphate buffer addition hydroxy methyl propyl cellulose. 1ml contains 155 mg tolonium chloride powder

#### Indications

Exclusively for disinfection of periodontal pockets and in cases of peri-implantitis with Laser 660 nm

### **Contraindications and side effects**

Known allergic reactions to any of the components of **LaserHF Paro-PDT** solution. At the concentration provided, **LaserHF Paro-PDT** solution produces no side effect other than a transient colour change in the dentine on the surface. This is removed by the washing process performed after disinfection of the lesion as described in the application below.

### **Caution**

**Contamination with saliva must be avoided to prevent reinfection of the lesion. Avoid contact with eyes but in such cases wash out immediately with water until eye is clear.**

### **Application**

#### **To treat periodontal pockets:**

Prepare the pocket by thoroughly scaling and root planning the surface of the tooth. Flush out debris and ensure that any major haemorrhaging is arrested. Ensure that the site is isolated from the saliva. Dispense **LaserHF Paro-PDT** solution directly into the pocket such that the pocket is full of solution and slightly overflowing. Agitate **LaserHF Paro-PDT** solution gently for 60 seconds with application canula to ensure effective penetration of the pocket depth. For the subsequent laser treatment with PDT/LLP modus following the instructions in the laser operating manual, introduce the fibre into the lesion and place it such that it is at the bottom of the pocket and approximately in the midline of the tooth. Irradiate for recommended time and recommended power in the laser operating manual. The fibre should be ever on the move. When ready, continue with restorative procedure.

#### **To treat peri-implantitis:**

Clean and debride the lesion in the normal manner. After completion and arresting of major haemorrhaging, place the solution directly into the lesion site and allow the pocket to fill with solution. Leave for 90 seconds to ensure that solution penetrates to the depth of the pocket. Agitate **LaserHF Paro-PDT** solution gently for 60 seconds to ensure effective penetration of the pocket depth. For the subsequent laser treatment with PDT/LLP modus following the instructions in the operating manual of the laser, introduce the fibre into the lesion and place it such that it is at the bottom of the pocket and approximately in the midline of the tooth. Irradiate for recommended time and recommended power in the laser operating manual. Following the instructions in the laser operating manual, introduce the fibre into the lesion and place it such that the fibre reached the bottom of the pocket. Illuminate the site for the recommended time and power in the Laser Operating manual. If the lesion extends around the implant treatment should be carried out on both mesial and distal aspects of the structure. Additionally the internal surface of the superstructure should be disinfected as well. Fill the internal surfaces of the superstructure with solution, leave for 90 seconds and using the fibre, position it just above the margin of the superstructure and irradiate the internal surface for recommended time and power in the laser operating manual.

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HAGER & WERKEN GmbH & Co. KG · Ackerstraße 1 · 47269 Duisburg, Germany